

Bebird Note 5 Pro Ear Cleaning Otoscope with Camera



How to connect the App

Note: Do not use the device unless you have successfully connected the device to your mobile phone through the Android/iOS App and are streaming live video footage.

1. Download the APP

iOS: Search for “bebird” in App Store

Android: Search for “bebird” in Google Play



OR scan the QR code to download the app



2. Open the cover / Power on automatically




3. Open the APP / Connect device WiFi




4. Connect product WiFi "bebird-Note5-xxxxxx"




5. Back to bebird APP / Click on "GO" to use it

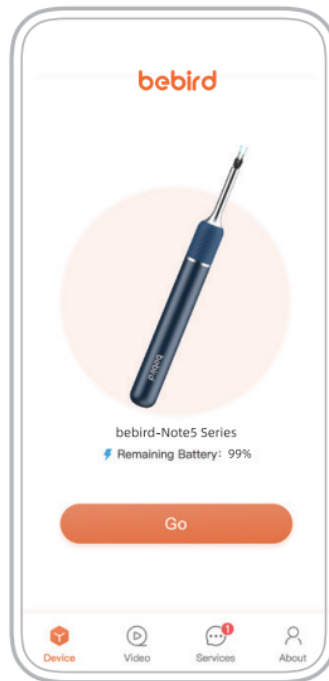


6. The visual screen

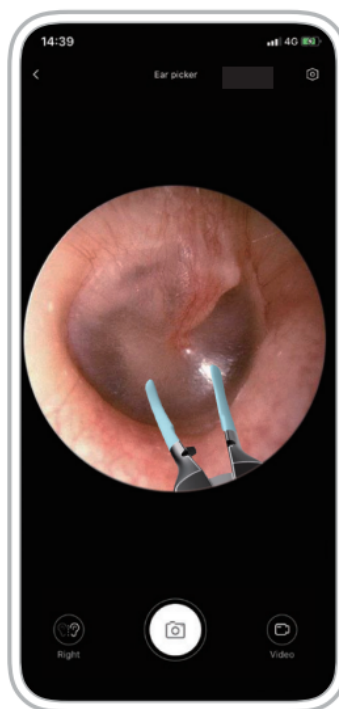

7. Press the button to control tweezers

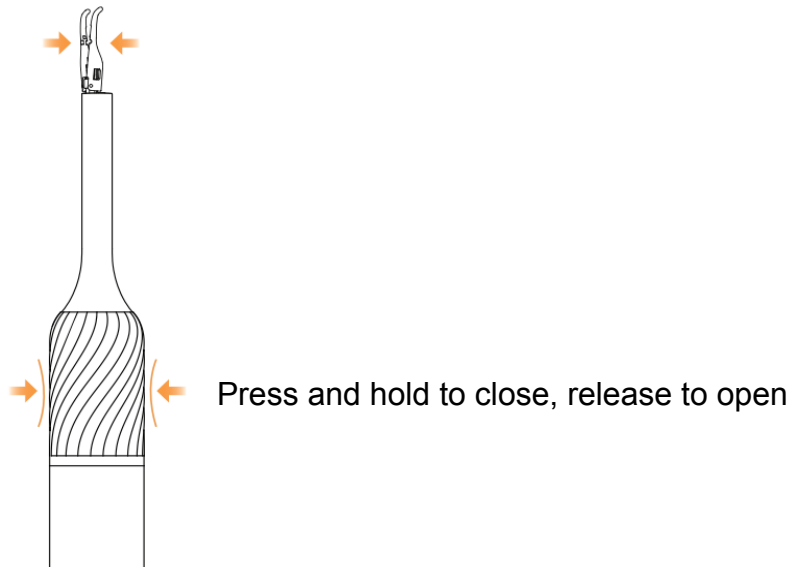


Product overview

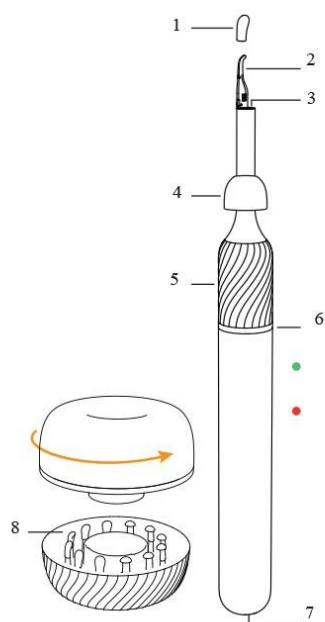


1. Silicone ear tip cover
2. Tweezers head
3. Lens
4. Ear canal protection plug
 - Move up and down to limit insertion depth
5. Tweezers switch
 - Press and control the tweezers to open and close
6. Indicator light
 - Green light flashes after starting
 - The red light keeps on during charge. After fully charged, the light is off.
7. Charging port
8. Standing base
 - Containing 12 pcs. ear spoons

How to use the tweezers

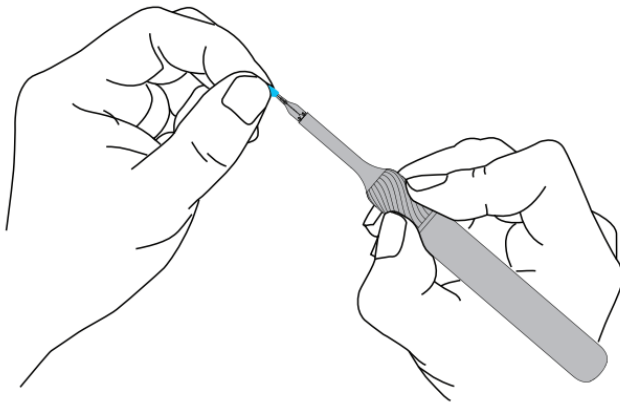


Hold the tweezers switch still with 2 fingers and remove the ear spoon with the other hand.

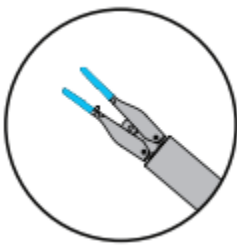

The tweezers open automatically when you release your hand.

How to replace ear spoon

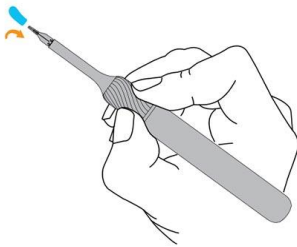


Press the tweezers switch first so that the tweezers are closed. Just put the ear spoon set on.

Note: The ear spoon needs to be placed as deep as possible on the tweezers to prevent it from falling off.

If the ear spoon is damaged, please stop using it immediately and contact us for replacement.

Accessories

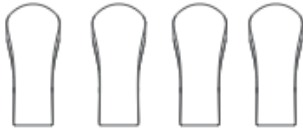

Spatula ear scoop x4 - Multiple ear scoop covers, use separately




Dome ear scoop x4 - Easily remove earwax in 360 degree



Sponge ear spoon cover x4 - Clean and comfortable, remove earwax and clean oil ear



Ear canal protection plug - A touch reminder to avoid reaching too deep in the ear canal

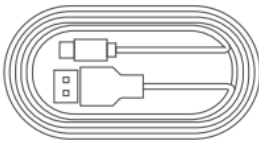

Charging cable



Alcohol swab x5 - Cleaning the lens or spoon



Double-ended cleaning brush - Suede head to wipe the lens; Brush head to clean the ear tip

Troubleshooting

No response after startup - Use the included charging cable to charge the device.

Camera LED stays on - Disconnect the device, restart it, and reconnect.

App closes quickly - Delete the existing app and scan the QR code to reinstall it.

Indicator light blinks quickly - Charge the device using the included cable.

Abnormal display in the app - Disconnect the device, restart it, and reconnect.

Wi-Fi connection is unstable or disconnects after a few seconds: Disconnect the device, restart it, and reconnect.

Blurred camera view - Clean the lens using a double-ended cleaning brush or an alcohol swab.

Image orientation is incorrect during use - Select the left or right ear mode in the app.

Note: If the problem continues after following these steps, contact Customer Service for further assistance.

Attention

1. Please use the equipped alcohol swab to wipe the lens carefully when cleaning.
2. The ear spoon cover must be installed properly, not loose before use to prevent it from falling off.
3. Please observe the surrounding environment before use. Do not use it in an environment that is susceptible to impact.
4. Prohibit children from using this product alone to avoid inadvertent injury.
5. Please do not put the device inside any form of liquid.
6. The battery must be charged once a month for effective use.
7. The temperature will rise slightly when using.
8. The product temperature will rise slightly when using (from bebird laboratory data, the maximum temperature of the whole machine does not exceed about 45°C). Please feel free to use.