

# **Bebird T15 Smart Visual Ear-Clean Ear Cleaner with Camera**




iOS Search for “bebird” in App Store  
Android Search for “bebird” in Google Play



Scan QR code to download the APP



Open the cover - Power on automatically

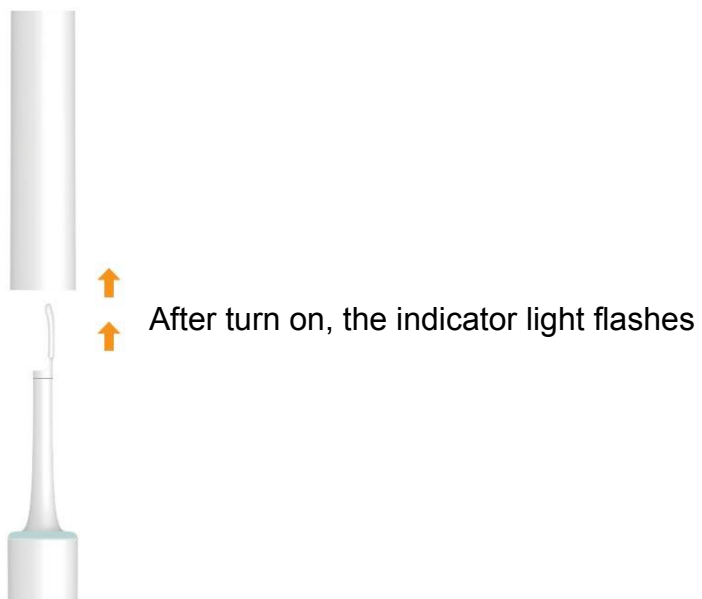


Open the APP - Connect device WiFi



## Connect WiFi

Connect product WiFi “bebird-T15-xxxxxx” / Back to bebird APP Click on “ GO ” to use it

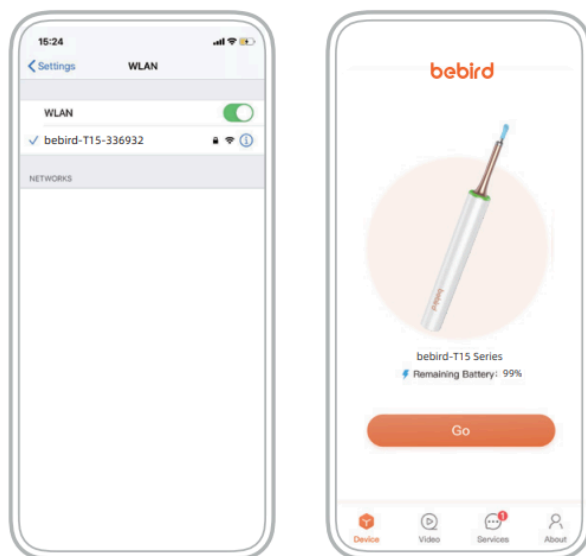

## Replace the ear spoon head

Rotate the ear spoon head, pull out and replace the ear spoon and fasten the ear spoon head.



Insert and fasten the ear spoon head

Assemble correctly before use to avoid dropping out

Charging - Red light always lights on

## Accessory



Metal & Silicone wrapped ear Spoon (Small ) x 1  
Harmless ear cleaning



Double-color Q-elastic ear spoon ( Small ) x 1  
Thorough removing of cerumen



Double-color Annular ear spoon x 1  
Easy Scraping; higher curvature



Double-color Q-elastic ear spoon ( Big ) x 1  
Easy and thorough removing of cerumen



Metal pimple squeezer rod x 1  
Easily squeeze out acne



Sponge Ear Spoon x 2  
Removing of earwax and grease in ears



Storage Bin x 1  
Store accessories inside



Charging Cable x 1  
TYPE-C Charging cable

## **Troubleshooting**

The product does not work after power-on?

- Please charge for 85 minutes with the standard USB cable.

The camera indicator is not lit?

- Power off the product and restart it.

Crash of APP?

- Uninstall the APP and scan the QR code to install it again.

The accessory holder cannot be installed smoothly?

- Rotate the ear spoon head, pull out and replace the ear spoon and fasten the ear spoon head.

The power indicator is lit in red?

- Please charge for 85 minutes with the standard USB cable.

After successful connection, the APP does not display normally?

- Disconnect WiFi between the mobile phone and the product, restart the product and connect WiFi again.

The WiFi connection is unstable. Sometimes, it disconnects just several minutes after connection.

- Restart the product and connect the APP again.