


# Alarm Clock for The Deaf and Hard of Hearing T1H

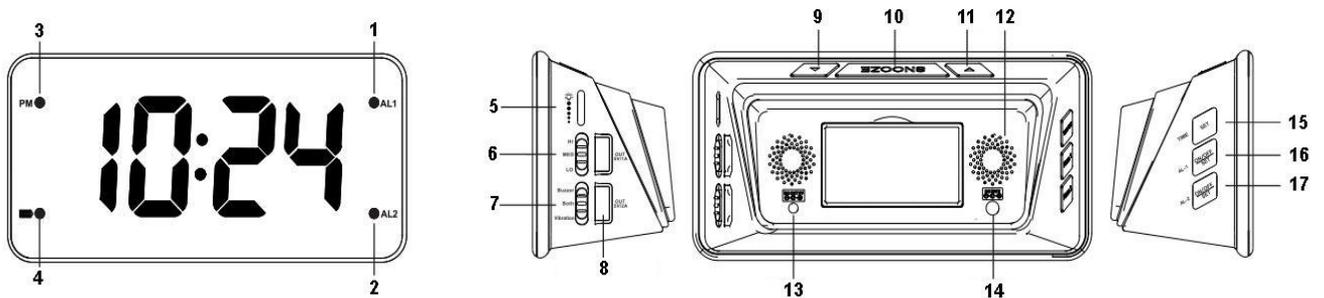

## Product information

1. Alarm1
2. Alarm2
3. PM
4. Battery indicator (blinks when battery power is less than 10%)
5. Dimmer
6. Buzzer volume / vibration intensity
7. Alarm mode
8. USB charging port
9. Up button
10. Snooze button
11. Down button
12. Buzzer
13. DC output 5V
14. DC input 5V
15. Time set
16. Alarm1 on/off/set (AL1)
17. Alarm2 on/off/set (AL2)



## Time setting

1. In time mode, press and hold the Time-Set button for 2 seconds to enter into time setting mode, the hour digits flash, then press the Up or Down button to set the current hour (hold to advance rapidly).
2. Press the Time-Set button again, the minute digits flash, then press the Up or Down button to set the minute.
3. Press the Time-Set button again, the 12h/24h digits flash, then press the Up or Down button to set the time format.
4. Press the Time-Set button again, the year digits flash, then press the Up or Down button to set the year.
5. Press the Time-Set button again, the month digits flash, then press the Up or Down button to set a month.

6. Press the Time-Set button again, the day digits flash, then press the Up or Down button to set the day.
7. Press the Time-Set button again, save and exit time setting mode.

### **Alarm setting**

1. In time mode, press and hold the AL1 button for 2 seconds to enter into alarm setting mode, the hour digits flash, then press the Up or Down button to set the hour (hold to advance rapidly).
2. Press the AL1 button again, the minute digits flash, then press the Up or Down button to set the minute.
3. Press the AL1 button again, the alarm day digits flash, then press the Up or Down button to set alarm day.
  - Single-day only (1-1)
  - Weekdays only (1-5)
  - Weekends only (6-7)
  - Daily (1-7)
4. Press the AL1 button again, save and exit alarm setting mode.

Setting for AL2 is the same as AL1.

### **Switching Alarm on/off**

- In time mode, press the AL1 button to switch on/off the alarm, the AL1 indicator will be on/off correspondingly. Switching on/off for AL2 is the same as AL1.

### **Stopping alarm when it goes off**

- When AL1/AL2 is sounding, the related AL1/AL2 indicator flashes. Press any buttons (except Snooze button) to stop and reset the alarm to go off the next day.

### **Adjusting snooze duration**

1. In time mode, press and hold the Snooze button for 2 seconds to enter into snooze duration setting mode and the digits flash. Press the Up or Down button to set the current duration (hold to advance rapidly). Default snooze duration is 9 minutes, you can adjust it from 1-15 minutes.
2. Press the Snooze button again, save and exit snooze duration setting mode.

### **Activating snooze when alarm goes off**

- When AL1/AL2 is sounding, press the snooze button, the alarm will suspend and go off again after the set snooze duration.

### **Alarm test mode**

- Press and hold the Up button for 3 seconds to enter alarm test mode, then you can set the right side switcher to select your favorite buzzer volume and vibration intensity. After that, short press the Up button to exit the test mode.

### **Alarm mode setting**

- Adjust the right side switcher to select your desired alarm mode: buzzer, buzzer & vibration, vibration

### **Adjusting brightness**

- Adjust to your favorite brightness level by turning the knob on the right side.